

Set-up Instructions - Attaching Additives to the STAT IV 1000

U.S. Patent No. 5,163,923 • Canadian Patent No. 2,119,376

STEP ONE

If an additive is to be given, first move any fluid containers on the high hooks to the center hook. Then hang the container of additive on a high hook of the GYRO Fluid Hanger.



STEP TWO

Remove the Blue cap from the top of the IV1000 drip chamber.



STEP THREE

Attach the male Luer connector on the Secondary I.V. Set (V1921) to the top of the drip chamber.



STEP FOUR

Remove the protective cap on the Secondary I.V. Set (V1921) piercing pin and the protective cap from your additive. Insert the piercing pin into the additive outlet port. Adjust additive flow with the roller clamp.



INTERNATIONAL WIN, LIMITED

340 North Mill Road, Suite 6, Kennett Square, PA 19348-2853 USA • (610) 444-0170 • (800) 359-4946 • Fax (610) 444-0171
e-mail: info@internationalwin.com • www.internationalwin.com